

PRODUCT DESCRIPTION

Shower Chair KY-1202

■ Safe use instructions

- 1. Please read the installation instruction manual carefully before ues, and use the product correctly and safely. Please keep the instruction manual properly for reference at any time.
- 2.After receiving the goods, please carefully check whether the product and accessories are complete, if there is any missing please be sure to contact us the first time.
- 3. This product should be placed in a dry, ventilated, stable, non-corrosive substances indoor.
- 4.Regularly check whether the product is intact every week to ensure safety. If there are excessive wear products and accessories and there are safety risks, please replace in time.

Product display



■ Basic product parameters

Installation instructions



Product line: Health product line Product series: Shower chair

Product name: Shower chair Product model: KY-1202

Product process: solderless process Pipe wall thickness: 1.3mm

Material: Aluminum frame +PE seat plate + non-slip rubber foot mat

Optional accessories: non-slip foot pad /45° bevel pad/blue cushion/grey cushion

Product size: Total height (73~83CM) / Total width (56CM) / Sitting width (40CM)

/Sitting height (43~53CM) / Sitting depth (33CM) / Back height (30CM) /Seat size (33*40*4.5CM)


This product is made of high strength high quality aluminum alloy material, surface oxidation treatment, smooth, delicate, beautiful and durable; Fixed with bolt, double pipe connection, tool free installation




(1) Align the upper and lower legs with the seat plate as shown. (Note: the middle shape of the upper leg is consistent with the direction of the groove of the seat plate).



102 Install the short screw with a wrench to the position shown.

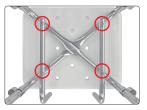

(3) Arrange the two backrest brackets as shown, align the holes, and install the four long screws and washers to the red circle using a wrench.

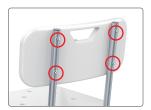

M In the red circle, align the backrest bracket with the backrest hole and install the four short screws and washers on it using a wrench.



05 Insert the handrail into the hole. (Note: shower holder free installation).



6 Adjust the four adjusting legs to the same height, install the upper and lower legs, and check whether the overall link is tight, installation is complete.

Product accessories display



Parcel list

- ① One piece for each leg
- ② Handrails 2 pieces
- 3 Backrest bracket 2 pieces
- 4 Back 1 piece
- ⑤ One piece of seat
- ⑥ Adjustable legs 4 pieces
- 7 5 short screws
- ® 4pieces of long screws
- 9 1 hex wrench
- ¹⁰ Shower stand 1 piece
- 11) 8gaskets

